FOR HOSPITAL USE ONLY • SOLO PARA USO EN HOSPITALES

# Boppy® HC Pillow for Newborn Feeding

## Durable • Wipeable • Comfortable

### Usage Instructions:

To maintain cleanliness between patients, please wipe the entire pillow thoroughly with a hospital grade disinfectant cleanser.

Allow the pillow to air dry 2-3 minutes before applying the disposable slipcover or before storing.

Please dispose of used slipcovers in the waste receptacle, or as appropriate.

Please replace pillow every 6 months.

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### ▲ WARNING

Possible Suffocation Hazard.

Never allow baby to sleep on the Boppy® pillow. Do not allow baby to lie face down on a Boppy® pillow. To prevent positional asphyxia, keep baby's airway open at all times. For proper breathing when using the pillow for propping, do not allow baby to be curled up in the center of the pillow area or be propped up too high on the pillow. Do not use in crib, cradle, bassinet, playpen, play yard. Not for use on a bed, either with the baby alone or with a caregiver who may fall asleep while feeding the baby. Do not leave baby unattended - adult supervision is required. Improper use of this product could result in serious injury or death. Baby must be supported by your arms holding the baby at all times. Do not use with baby while standing.

**IMPORTANT:** Remove all packaging and fasteners before use.

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